

WARNING SIGN ARTWORK INSTRUCTIONS

If you want to retrospectively use this Warning Sign on your chairs, please use the artwork contained on page 2 this PDF and produce yourself using the recommendations below.

- 1. *staxi*[®]** recommend using rivets to attach the warning sign to the chair.
- 2.** Artwork shows trim marks and 6 drill holes.
- 3. *staxi*[®]** recommend the following printing specification: Inkjet print 4/0 on a 5mil white film with a 3mil gloss lamination mounted to 0.060 PetG plastic.



WARNING

To reduce the risk of **SERIOUS INJURY** or **FATALITY** follow these rules for safe use of this *staxi*[®] Transport Chair:



Passengers must enter this chair from the side

- Raise the armrest to enter from the side, never from the front of the chair. See handlebar label.



Passengers must be able to sit upright unassisted

- Sit upright, fully back in seat with armrests lowered and feet on the footrests.
- Do not use this chair for young children or those without upper truncal control. Passengers must be able to hold themselves upright unassisted.

Attendant must stay with chair when occupied

- Never leave a passenger unattended. Passengers are unable to move the chair and are at risk in the event of an emergency.

Do not overload the chair

- The total weight limit for this chair is 500lbs. This includes the weight of the passenger plus any bags, cargo, or accessories loaded onto the chair.



Avoid tip-overs and falls from the chair

- Only one (1) passenger at a time.
- Avoid going over curbs or ledges. If you must, *staxi*[®] recommends going rearward by carefully lowering the rear wheels over/onto the obstacle then backing over in a controlled manner.
- Do not attempt a 'U' turn or a 360° maneuver on a slope, ramp or incline.
- Do not stand on the footrests. Footrests are only to be used by a properly seated passenger.
- Never wheel this chair on escalators or stairs.

Always use seatbelts (if installed)

- Always use the seatbelt, if one is provided. For more information on seatbelt options, contact *staxi*[®].

Use caution when maneuvering chair

- As with all transport chairs and wheelchairs, extra care should be taken when maneuvering an occupied *staxi*[®] chair through narrow or crowded spaces. Be careful not to pinch a passenger's arms or hands between the armrests and walls or door edges when navigating a narrow doorway or tight gap. Take care that the passengers feet are not jammed between, or stubbed on, objects to the front or sides of the chair.
- Use care and avoid collisions with other people when using a *staxi*[®].

For transporting passengers only

- Do not transport luggage in the seat or on the passenger's lap. Items may fall on slopes. Use the luggage area below the seat to carry baggage.
- Never place children in car seats on this chair.

Check functional parts regularly

- Inspect functional parts of the chair monthly. See the Maintenance section of the owner's manual for recommended checks.

Before using, read the OPERATOR'S MANUAL.
Please download at: www.staxi.com



Para obtener información de seguridad en español, visite nuestro sitio web.

Pour plus d'informations de sécurité en français, visitez notre site Web.

Wow-Care™

Toll Free Phone: 1-866-969-1463
or 1-866-WOW-1463

Email: Wow-Care@staxi.com

Website: www.staxi.com

staxi[®]

Rolling out Excellence