



Product	Aisle Chair AC020
---------	-------------------

European Authorized Representative	Manufacturer
MedEnvoy Global B.V. Prinses Margrietplantsoen 33 – Suite 123 2595 AM The Hague The Netherlands	Staxi Corp 424 Williamson- Zebulon Rd Williamson GA 30292 USA

1.0 Classification and Intended Use

Staxi Aisle Chair is a class I medical device under Regulation (EU) 2017/745 (for MDR). EMDN Code: V080502.
The Staxi Aisle Chair is intended to be used in the transport of non-ambulatory passengers from their personal mobility assistance equipment to their seat on an aircraft, and vice-versa, under the direction of 2 attendants. Typical use involves a single transport episode of transient duration (less than 1 hour).



Fig 1. Staxi Aisle Chair

2.0 Chair Part Codes – incl. Colors

AC020-X			
---------	---	--	--

3.0 Instructions for safe use:

- Before use, visually inspect the chair to confirm that all components are securely fastened and 4 wheels roll smoothly. Then, ensure that the brake is functioning properly by testing that the chair will not roll forward while the brake is engaged.
- To be used transporting passengers from aircraft door to seat only.
- If transfer process calls for crossing a ledge or gap into the aircraft, use the front and rear lift bars to cross the gap.
- Always use two attendants to perform transfers, one at the front and one at the rear of the transport chair.
- Attendants are to ensure the chair does not move during the transfer
- Transfer passenger as per the responsible companies 'Transfer SOP'
- **Harness, both thigh straps, and lower leg strap must be secured before any movement.** Do not use if any of the above is missing or defective
- To move chair
 - Disengage Yellow Brake Stop Bar
 - Disengage the Handle Bar Brake (Moving): Squeeze brake bar and push.
 - Engage the brake (Stopping): Release brake bar.
 - Yellow Brake Stop bar will auto-engage upon engagement of brake.

- **Do not disengage Yellow Brake Bar or Handle Brake Bar while transferring the passenger.** Ensure harness and all straps are securely fastened before disengaging any element of the braking system
- Weight Capacity: 500lbs / 227kg.
- Keep the chair away from open flames and sources of intense heat.
- Attendants must stay with chair when occupied.
- Do not overload the chair.
- Avoid tip-overs and falls.
 - Only one passenger at a time.
 - Do not attempt a U-turn or 360 maneuver on a slope, ramp or incline.
 - Do not stand on footrests.
 - Never wheel this chair on escalators or stairs.
- Use caution when maneuvering the chair, to avoid pinching arms or hands between armrests and walls or door edges when navigating narrow doorways or tight gaps.
 - Avoid tight spaces.
- Exclusively for use when passengers require assistance from their personal wheelchair to the aircraft seat. Designed to ease egress down the airplane aisle. Do not use for general transport from point-to-point in the airport.
- Never place children in car seats on this chair.

4.0 Handling and Storage:

- Do not stack chairs on top of each other.
- Keep chairs indoors where possible, protected from weather to maximize lifespan.

5.0 Cleaning:

- Clean between users or when soiled using a neutral-pH, non-corrosive detergent or approved low-level disinfectant; follow the cleaning product usage label for dilution/contact time.
- Wipe only; do not immerse. Avoid solvents, abrasives, steam/pressure washing, and unapproved corrosive agents.
- After cleaning, inspect critical parts; remove from service if damaged or not functioning.

6.0 End-of-Life Disposal Information:

Dispose of the chair and its components in accordance with local regulations. End-of-life components may be managed through normal solid-waste or recycling streams. Cushion and textile parts may be recycled where programs exist.

7.0 Declaration:

	<p>Staxi Corp declares that the above-mentioned products meet the provisions of the following EU legislation:</p> <ul style="list-style-type: none"> • Medical Devices Regulation (EU) 2017/745 • EN ISO 15223-1:2021 for symbols • EN ISO 20417:2021 for labelling
---	--



8.0 Notice:

If you observe or have a serious incident with the Staxi Aisle Chair, report this to Staxi and the and the competent authority of the Member State in which you are based.